



## 1B-419

Eosin AG

## 1B-421

Eosin BA

## 1B-423

Eosin, soluble in alcohol and water

## 1B-425

Eosin, yellowish, water soluble

## 1B-427

Eosin, pure

In-vitro diagnostic agent

### Description

The products 1B-419, 1B-421, 1B-423, 1B-425 and 1B-427 are ready-to-use solutions for professional users in histology.

The product comes in 4 different pack sizes: 1B-419.00010 (10g bottle), 1B-419.00025 (25g bottle), 1B-419.00100 (100g bottle) and 1B-419.01000 (1kg bucket)

The product comes in 4 different pack sizes: 1B-421.00010 (10g bottle), 1B-421.00025 (25g bottle), 1B-421.00100 (100g bottle) and 1B-421.01000 (1kg bucket)

The product comes in 7 different pack sizes: 1B-425.00010 (10g bottle), 1B-425.00025 (25g bottle), 1B-425.00100 (100g bottle), 1B-425.00500 (500g bottle), 1B-425.01000 (1kg bucket), 1B-425.05000 (5kg bucket) and 1B-425.15000 (15kg drum)

The product comes in 10 different pack sizes: 1B-425.00010 (10g bottle), 1B-425.00025 (25g bottle), 1B-425.00100 (100g bottle), 1B-425.00500 (500g bottle), 1B-425.01000 (1kg bucket), 1B-425.05000 (5kg bucket), 1B-425.10000 (10kg drum), 1B-425.15000 (15kg drum), 1B-425.20000 (20kg drum) and 1B-425.25000 (25kg drum)

The product comes in 4 different pack sizes: 1B-427.00010 (10g bottle), 1B-427.00025 (25g bottle), 1B-427.00100 (100g bottle) and 1B-427.01000 (1kg bucket)

### Main components

Eosin Y (C.I. 45380)

### Purpose

The staining solution Eosin is used for cell diagnostics for the examination of histological samples (e.g.



histological sections). The eosin staining solution is used to counterstain proteins, connective tissue, fibres and keratin after a core staining with haematoxylin. It is a ready-to-use dye solution for professional users. It can be used individually or in combination, for haematoxylin-eosin overview (H&E) staining routinely used in histology.

### Sample material and sample preparation

Sampling may only be carried out by qualified personnel. All samples must be processed with state-of-the-art technology. All samples must be clearly labelled.

Sample material: Sections of human tissue after fixation for instance, by means of buffered formol and fixation mixtures with ethanol and formalin and subsequent embedding in paraffin or frozen sections, as well as smear specimens.

### Test principle

Together with haematoxylin, eosin is important for H&E overview staining. Eosin is regressively stained, correspondingly first overstained and then differentiated. The negatively charged dye eosin binds to the positively charged plasma proteins. Cytoplasm turns pink-red, while erythrocytes appear orange.

### Staining

H&E staining is based on two individual stains. Haematoxylin is used for staining the cell nuclei and eosin for staining the cell plasma proteins. Before staining, deparaffinise the sections and transfer them to distilled water via a descending ethanol series. The samples are rinsed in tap water after staining with haematoxylin. Rinsing with hydrochloric acid solution (0.1%) is used for differentiation; by increasing the pH value, the cell nuclei in particular turn blue. This is followed by staining with eosin solution. The samples are rinsed once again in tap water and then transferred to xylene via an ascending ethanol series. The samples can be covered with a synthetic covering medium for subsequent microscopy.

To ensure the differentiability of the target structures, suitable control specimens should be kept along with the staining.

The usual staining protocols known from literature must be used.  
Staining may only be carried out by qualified personnel.

### Result

Cytoplasm	pink-red
Erythrocytes	orange

### Precautionary measures

When removing the product, care must be taken to avoid contamination of the storage vessel. Once the solution has been removed, it must not be returned to the canister. If turbidity or solids appear, discard the product. The product is intended for single use and must not be reused.

### Storage and shelf life

Store the unopened containers in a dry place at 15 to 25 °C, avoiding direct sunlight.

The shelf life is 2 years. See also the best-before date (BBD) on the label. Once the containers have been opened, the shelf life corresponds to the best-before date, as long as the storage conditions are observed and the solution is handled properly.

### Safety notice

If any serious incidents occur in connection with the product, please report them to the manufacturer and the national authority.



## Literature

Romeis, Mikroskopische Technik, Editors: Maria Mulisch, Ulrich Welsch, 2010, Springer Spektrum, 18th edition



Waldeck GmbH & Co. KG  
Havixbecker Str. 62  
48161 Münster  
Deutschland  
Amtsgericht Münster  
HRA 1926

Komplementärin:  
Wahlig Beteiligungs- und  
Verwaltungsgesellschaft mbH  
Amtsgericht Münster  
HRB 692

Geschäftsführer:  
Wilhelm Wahlig  
Ingo Gerdes

Bankverbindungen:  
SPK Westmünsterland  
IBAN:  
DE49401545300036060184  
BIC:  
WELADE3WXXX

USt.-Id.-Nr.:  
DE126046718  
Steuer-Nr.:  
5336/5779/0252  
Gläubiger-ID:  
DE67ZZZ00000021849

Kontakt:  
Tel. +49 2534 970 0  
Fax +49 2534 970 258  
info@waldeck-ms.de  
www.waldeck-ms.de